Ref. 302800 / 302810

DEUTSCH

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Ref. 302800 / 302810

ENGLISH

Cattle Crush A 8000 and A 8000 for caesarean section

Behandlungsstand A 8000 und A 8000 Kaiserschnitt / Cage de contention A 8000 et A 8000 césarienne /
Behandelbox A 8000 en A 8000 keizersneden / Caja de contención A 8000 y A 8000 cesarea

Additionally required tools for assembly:

- Wrench set
- Allen wrench
- Ladder

Mounting with two persons in approx. 1 h

Installation and mounting Instructions


Included in delivery:

	Qty	Description
1	1x	Cattle Crush with parallel squeeze system
2	1x	Rump bar
3	1x	Operating lever
4	1x	Hand pump lever
5	1x	Head-chain-fixation

Accessories (optional):

	Qty	Description
1	1x	Baulk Gate, single-leaf (Ref. 302308)
2	1x	Baulk Gate, double-leaf (Ref. 302307)
3	1x	Hoof Care Kit (Ref. 302360)
4	1x	Head Fixation (Ref. 302408)
5	1x	Weighing System (Indicator (e.g. Ref. 480023) + Load Bars (Ref. 400500))

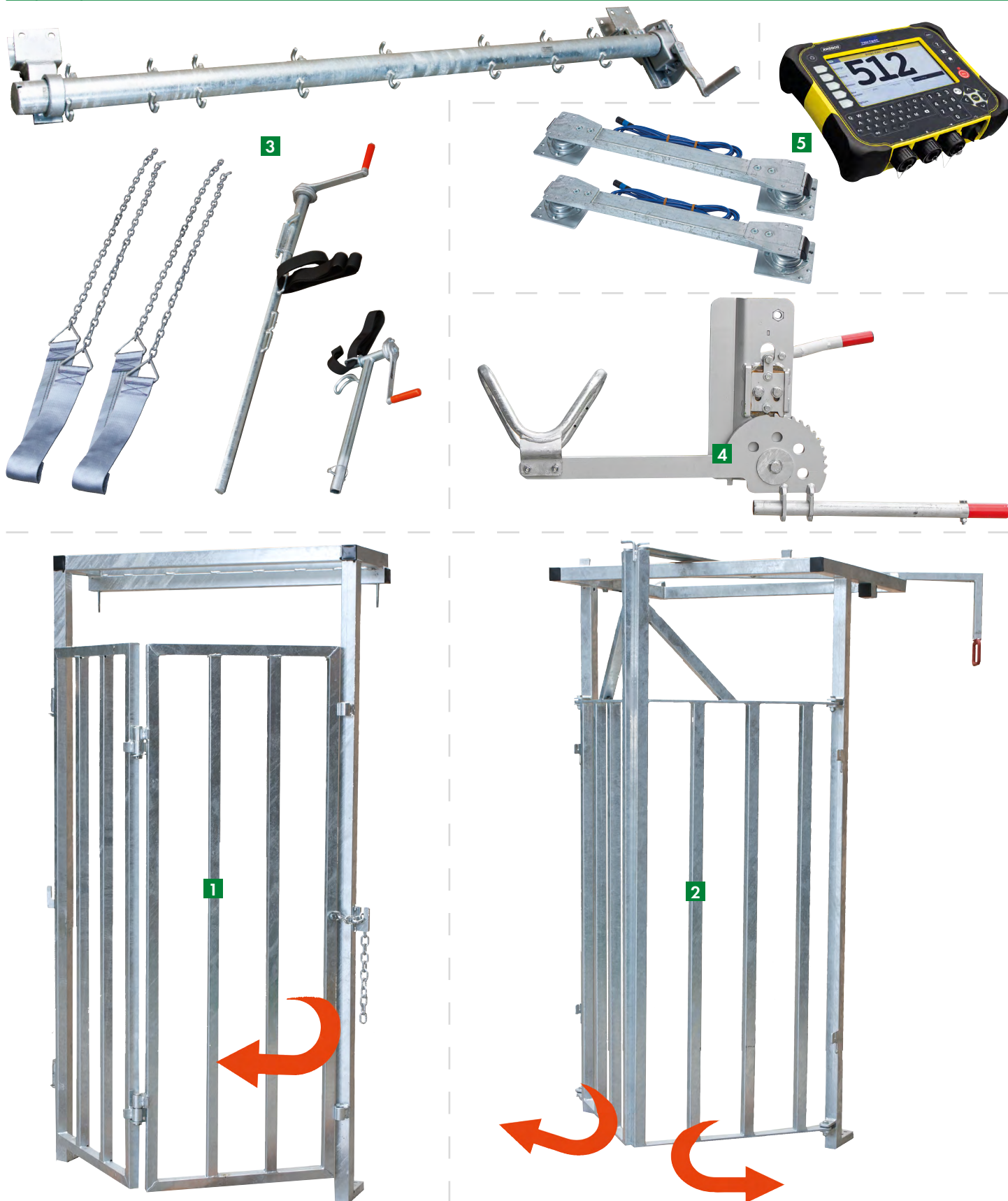Installation and mounting Instructions

- The product is completely pre-assembled.
- All operating elements can be mounted on both sides.
- Before each use, lubricate the sliding rod of the brake with penetrating oil.

- **Quiet environment makes it easier to work with animals. Try to guide or entice the animals rather than to drive or push them.**
- **Wear protective clothes such as gloves, security shoes and appropriate safety goggles when using an angle grinder.**

- 1.** Transport safely via palett fork.
Put it on level ground at the end of the working alley.

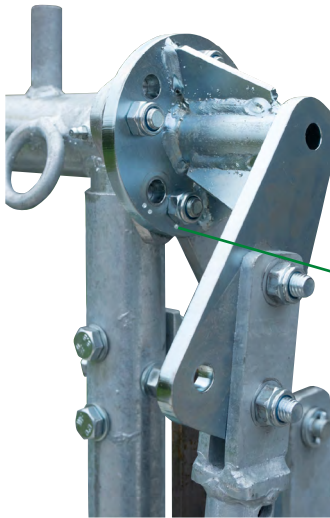
- 2.** For efficient and safe handling:
Professional handling systems with holding pen (see PATURA main catalog chapter „Handling Systems“).
Use Treatment Cage XL (Ref. 310106) for safe treatment and as a passage from working alley to the crush.

3. Installation and Operation Headbail Unit

A Installation Headbail Unit

Operation from the left:

For mounting the adaptor part, you have to use the hole with one dot for the lower screw. Make sure that the screw threads show to the outside.

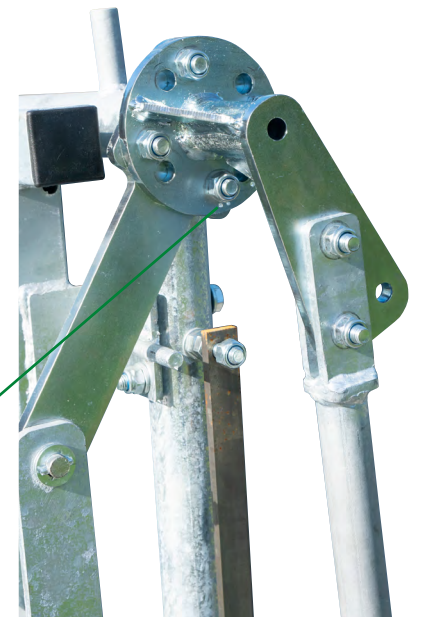
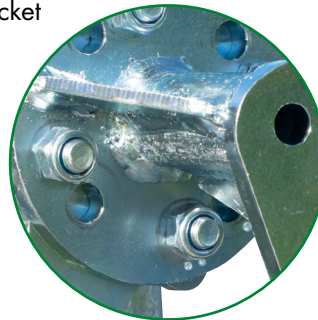
Attach the operating lever **3** to the bracket of the adaptor part.

Operation from the right:

For mounting the adaptor part, you have to use the hole with two dots for the lower screw. Make sure that the screw threads show to the outside.

Attach the operating lever **3** to the bracket of the adaptor part.

B Open head bail **3** just as wide as needed so that the animal comes straight through with its head. Guide animal in the crush and lock back door.

When guiding the animal in, one person must hold the operating lever **3** of the headbail in open position.

Installation and mounting Instructions

4. Catch the animal by operating the fully adjustable headbail. Make sure you lock it as tight as possible, so that the animal can not back up.

Treatments around the head (ear tags, nose rings etc.):

Open baulk gate **1** respectively **2**.

If necessary, use head-chain-fixation **5**.

Hook the chain into the lug **A** and lead it over the mouth. Pull trough the ring **B** upwards over the animal's head to the lug **C** pull tight and hook it into the lug **C**.

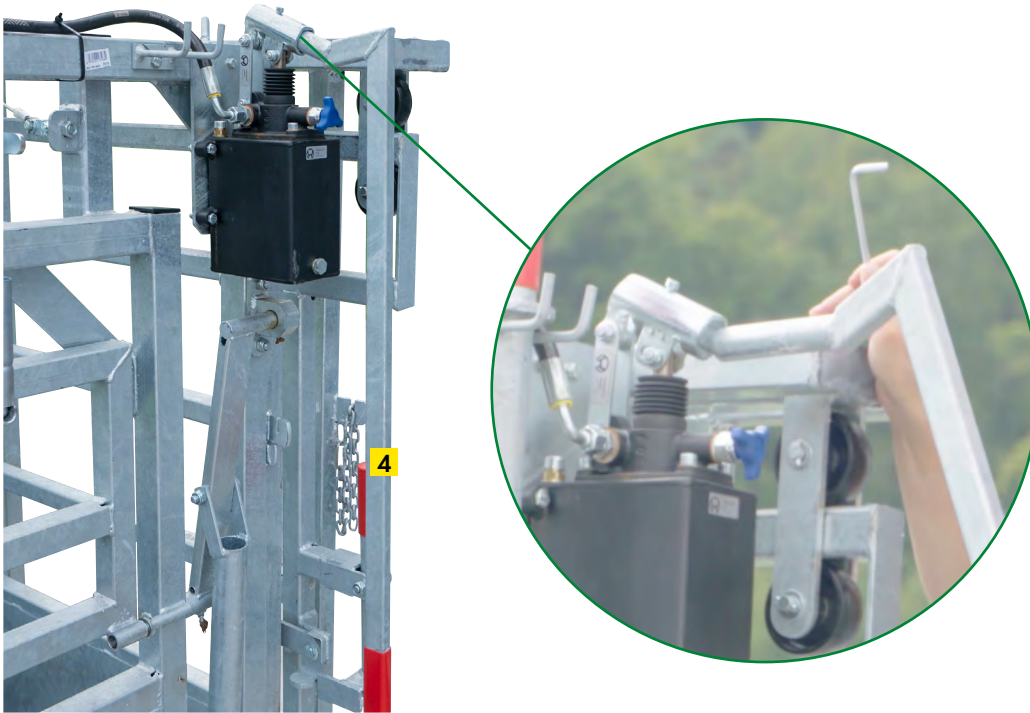
5. For safe fixation of the animal inside the crush:
Use the rump bar **2** in the guiding and push forward as much as possible. Fully secure the rump bar with an angle steel.

6. Installation Hand Pump Lever + Operation Parallel Squeeze System

A Mount the hand pump lever **4** into the pipe sleeve provided for using the parallel squeeze system.

B Fix the animal tightly by squeezing via a pumping motion **4**, release screw **D** has to be in a locked position. Return to the starting position by loosening the release screw **D**.


Installation and mounting Instructions

7. Installation + Operation Baulk Gate 7 (optional)

A Installation baulk gate, double-leaf (Ref. 302307)

Included in delivery:

	Qty	Description
2	1x	Baulk gate, double-leaf
6	1x	Operating lever for baulk gate
7	2x	Flat bar

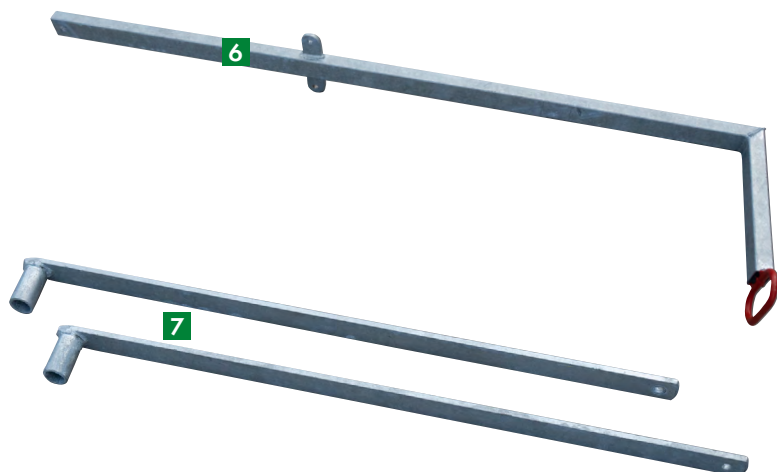Attach the frame of the baulk gate **2** to the cattle crush.

Fix the operating lever **6** to the frame of the baulk gate **2**.

Insert the pipe sleeves of the flat bars **7** into the upper wings of the baulk gate **2** and fix all with a screw on the bracket of the operating lever **6**.

Attach the lower wings of the baulk gate **2** in the holes provided in the bottom of the frame.

B Close the baulk gate before guiding the animals into the cattle crush.
(In case of double-leaf baulk gate **2**, secure the operating lever **6** behind the lock or, in case of the single-leaf baulk gate **1**, secure the chain!)

8. Treatments around the head (ear tags, nose rings etc.): Open baulk gate **1** respectively **2**.

Open lower doors or upper door to work on other parts of the animal.

Model A 8000 Caesarean section: right side, open the two lower doors or the upper flap.

left side, opening of the 5 specially arranged doors, for Caesarean section and other treatments.

Release animal:

1. Take out rump bar **2**
2. Keep back door locked
3. Open baulk gate **1** respectively **2**
4. Open the headbail unit using the operating lever **3** and hold it open until the animal has left the cattle crush


- Remove the cattle crush from the pasture after each use. Animals could damage it from the outside.
- Make sure it gets cleaned deply after each use and store it in a dry place.
- To properly secure the function of the crate for a long time, it is important to maintain it using quality lubrication on all lubrication nipples.
- Use only penetrating oil to maintain the headbail breaks.
- If the brakes are not in use for a longer period of time, wrap an oily cloth around the brake slide rod.

Installation and mounting Instructions

9. Installation + Operation Hoof Care Kit 3 (optional)

Included in delivery:

	Qty	Description
8	1x	Belly strap winch
9	2x	Belly straps
10	1x	Front foot winch
11	2x	Foot straps
12	1x	Hind foot winch

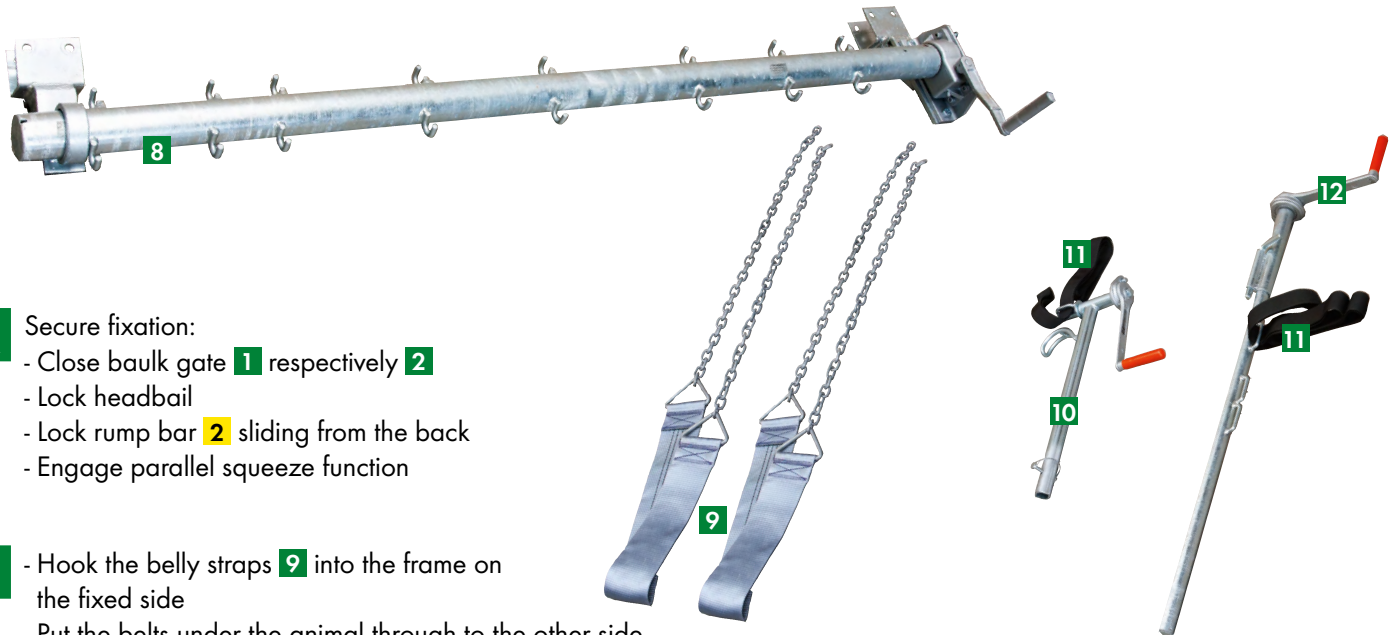- A** Secure fixation:
- Close baulk gate 1 respectively 2
 - Lock headbail
 - Lock rump bar 2 sliding from the back
 - Engage parallel squeeze function

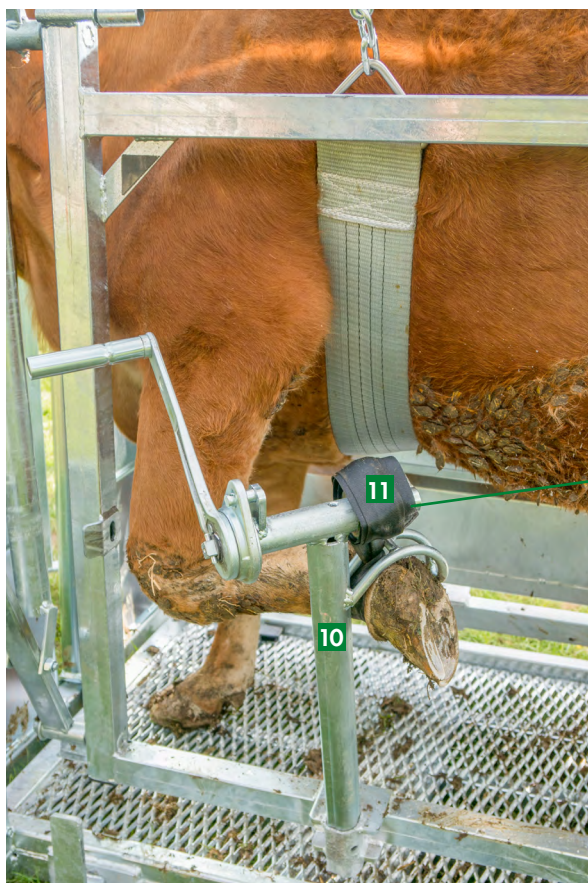
- B**
- Hook the belly straps 9 into the frame on the fixed side
 - Put the belts under the animal through to the other side
 - Hang into the hooks of the belly strap winch 8 and tighten the belly straps slightly


Make sure not to lift the animal too much to avoid it getting nervous!

- C**
- Connect front foot winch **10** by locking with a splint pin
 - Wrap strap **11** around the front foot (between dew claw and claw)
 - Guide strap **11** through the inside of the front foot support and put into winch **10**
 - Pull the foot upwards by turning the winch **10**, until it is firmly supported in the front foot rest

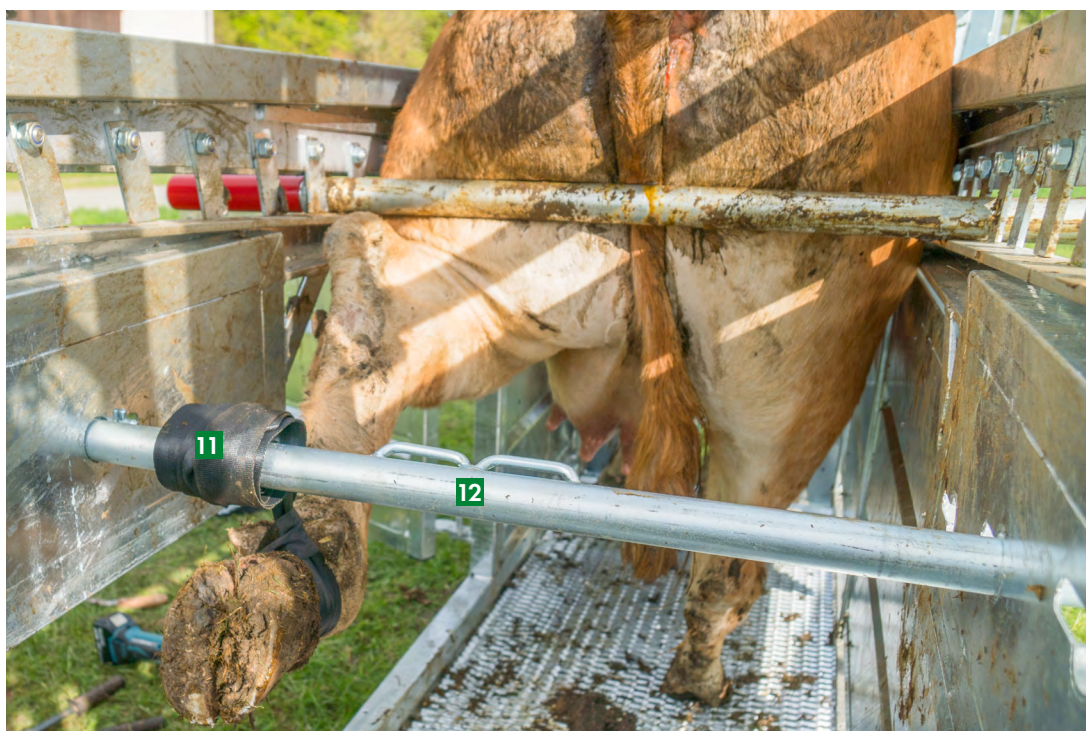- D**
- Lock hind foot winch **12** according to animal size in forseen openings above the back lateral door
 - Pull strap **11** around the rear foot (between dew claw and claw) and insert into winch **12**
 - Pull foot by turning the winch **12** upwards until it is firmly attached to the bar



Installation and mounting Instructions

10. Installation + Operation Head Fixation 4 (optional)

A Screw the head fixation 4 to the frame of the single-leaf baulk gate 1 using the U-profiles. Mounting the head bracket in the outer hole pattern.

Included in delivery:

	Qty	Description
4	1x	Head Fixation
13	1x	Operating lever
14	1x	Release lever

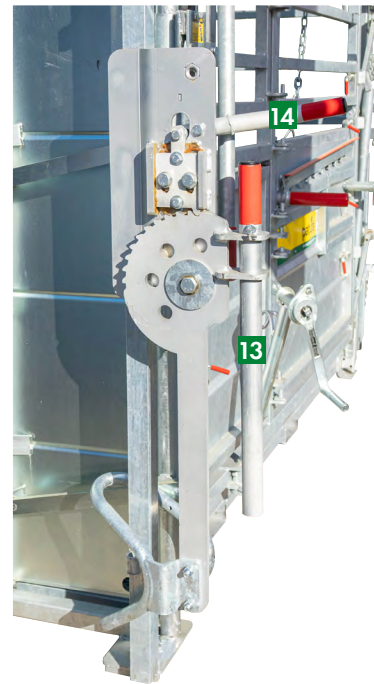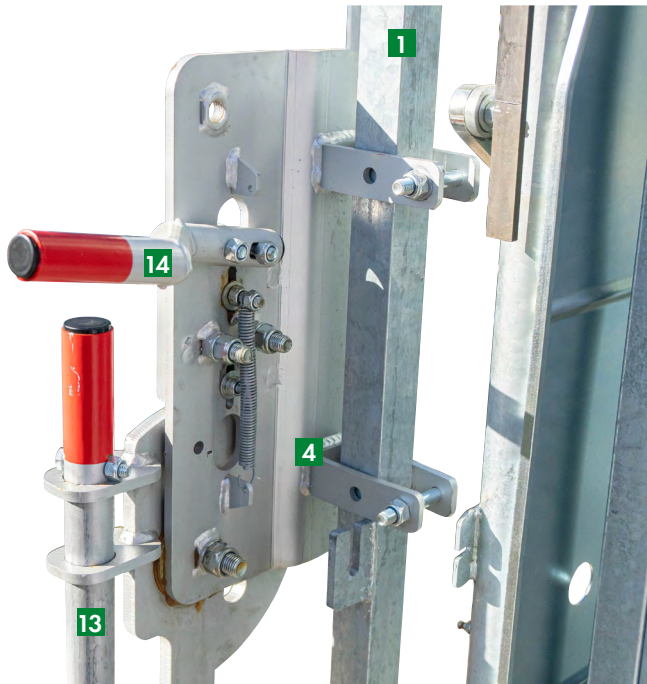This is an option for the single-leaf baulk gate only.

B After the animal has been fixed in the headbail, pull the operating lever 13 upwards from the rest position, then press down to bring the head fixation from below up to the head of the animal.

C Press the release lever **14** downwards to release.

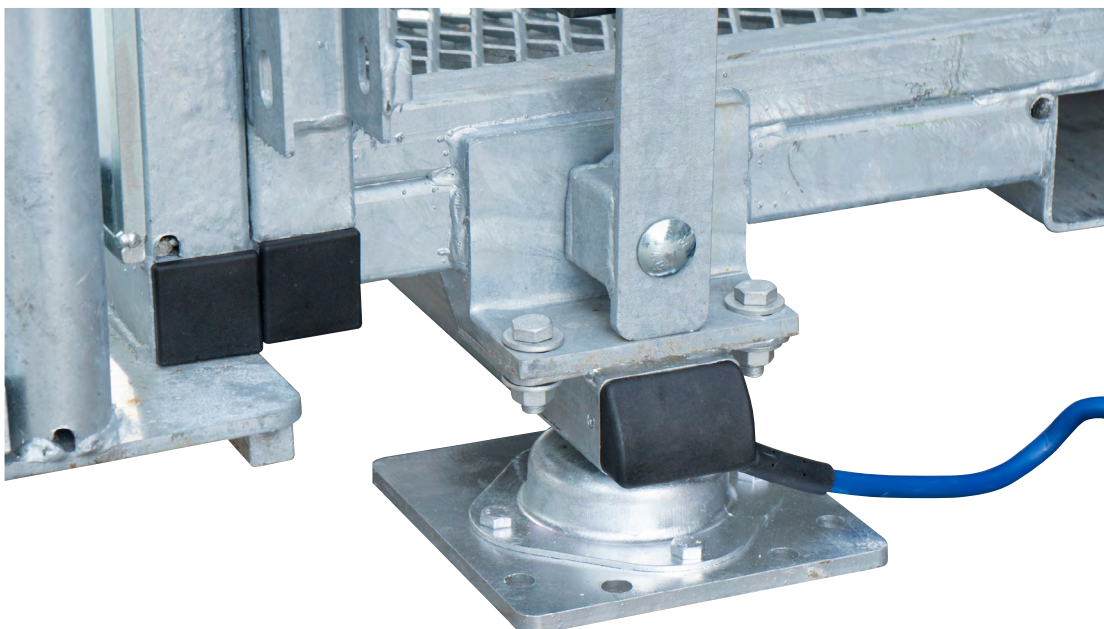
Before doing so, make sure to push the operating lever **13** back in!

11. Installation Weighing System 5 (optional)

- A** Lift the cattle crush safely and attach the load bars to the integrated mounting brackets of the crush. Do not tighten the screws yet!
Secure the cables of the load bars and place the crush to an even surface.
When all 4 feet of the load bars are full in contact with the ground, the screws can be tighten. Use the enclosed anchor bolts for stationary use.

- B** Attach the indicator with the enclosed mounting material to the cattle crush and fix the cables to safe them from damage.

PATURA Cattle Crush A8000

The universal cattle crush for all intended purposes

- All animals of one herd, from young stock to breeding bull, can be restrained and handled via the infinitely adjustable parallel squeeze system
- Young stock cannot turn around inside the crush
- Hoof care for small and large-framed cattle is possible at anytime
- The large inside compartment allows a fast and stressfree work process for man and animal
- The cattle crush has been designed for a large throughput of stock

10 advantages that will convince you

Advantage 1: Parallel Squeeze System (85 cm – 40 cm)

Advantage 2: Hoof Care Equipment with Recoilless Winches

Advantage 3: 2 Baulk Gate Options

Advantage 4: Headbail Units (12 cm – 85 cm)

Advantage 5: Steel Mesh Floor


Advantage 6: Optimum access to the animals from both sides

Advantage 7: Rump Bar

Advantage 8: Functional inside length of 2.75 m

Advantage 9: Wide open front section

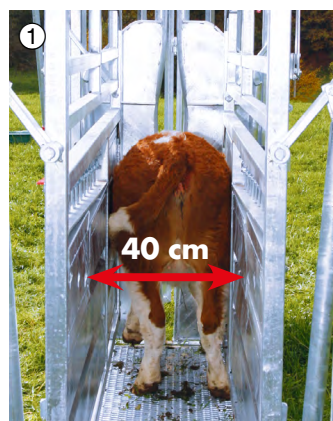Advantage 10: Simple transport via pallet fork

Parallel Squeeze System for all animal sizes

Optimal traction due to steel mesh floor

Side gates for easy access to all body parts

Animal restraint from the rear with rump bar

The cattle crush for professional herd management

Work efficiently, check the state of health, weight and weight gain of your animals regularly.

An important part of professional herd management is an electronic animal weighing unit which can be combined with a crush and which will store the date of each animal with the aid of its ear tag number.

② Hoof care equipment with recoilless winches

③ Baulk gate for easy and safe capture

④ Headbail to restrain all animal sizes. Head-chain-fixation included.

⑧ Inside length (2.75 m) enables a separation of the work process "entering" and "restraining"

⑨ Release of the animals through the wide-open front section

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