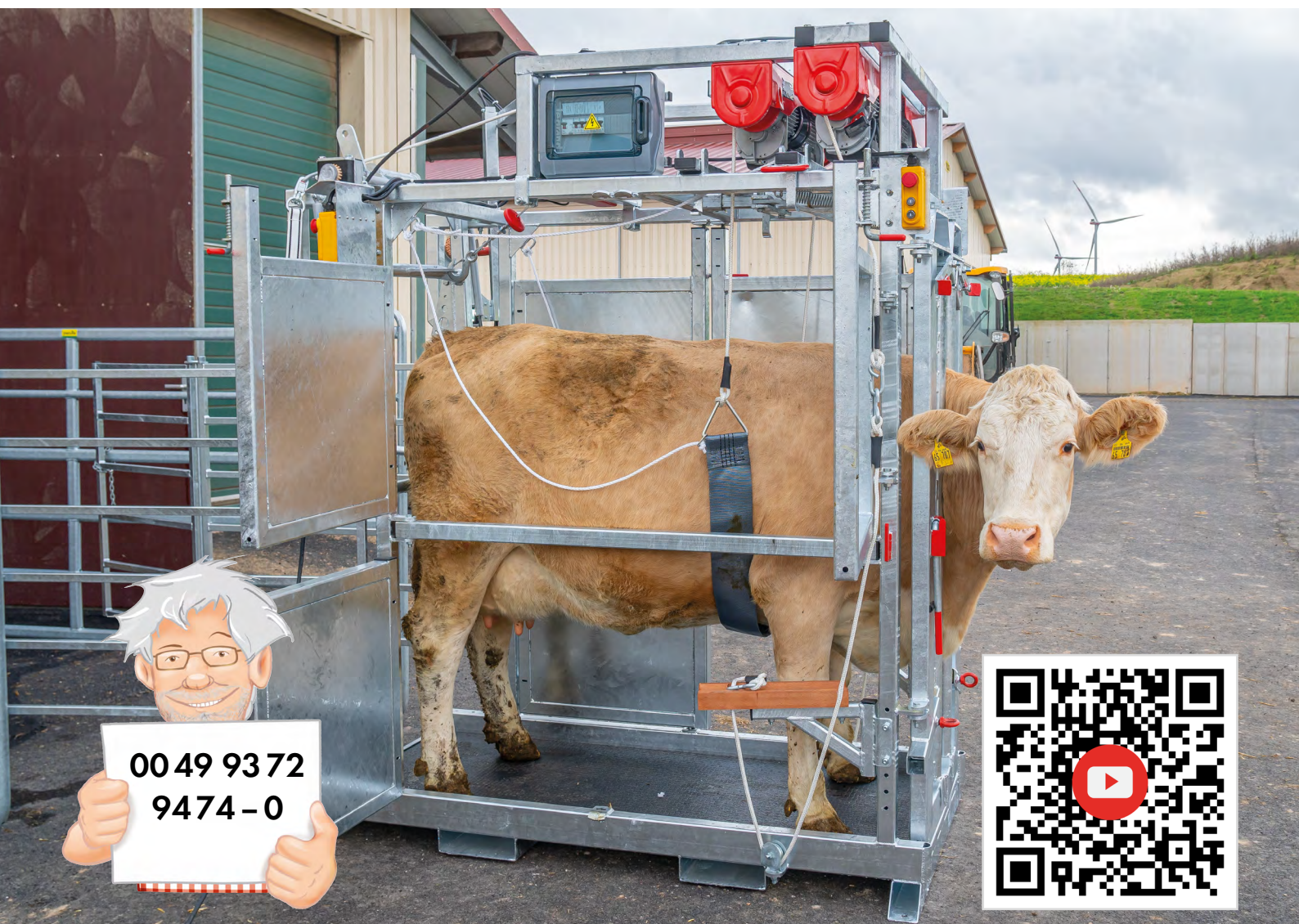
Ref. 310030

DEUTSCH

## Klauenpflegestand Profi E

Cage de parage électrique Pro / Hoof Care Crush Profi E / Klauwbehandelbox Profi E /  
Jaula Profi E para el tratamiento de pezuñas

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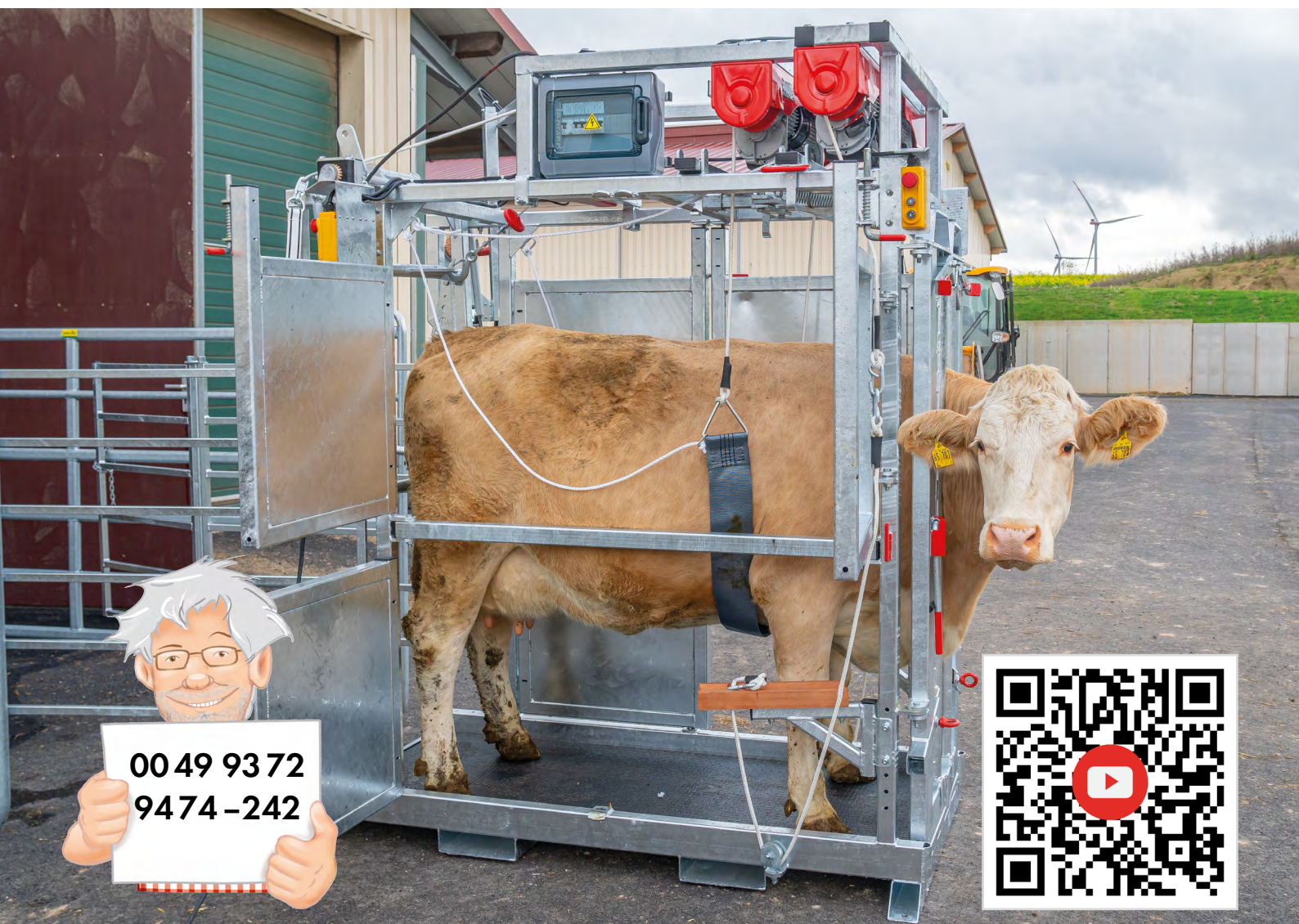
Additionally required tools for assembly:

- Wrench set
- Cable drum

Mounting with one person in approx. 30 min

# Mounting Instructions

## Included in delivery:

	Qty	Description
1	1x	Hoof Care Crush Profi E
2	2x	Front foot rest
3	1x	Belly strap
4	1x	Hook rear foot winch

## Dimensions:

Internal dimensions:	Length 190 cm, Width 82 cm, Height 180 cm
External dimensions:	Length 200 cm, Width 150 cm, Height 240 cm
Weight:	620 kg

## Technical details:

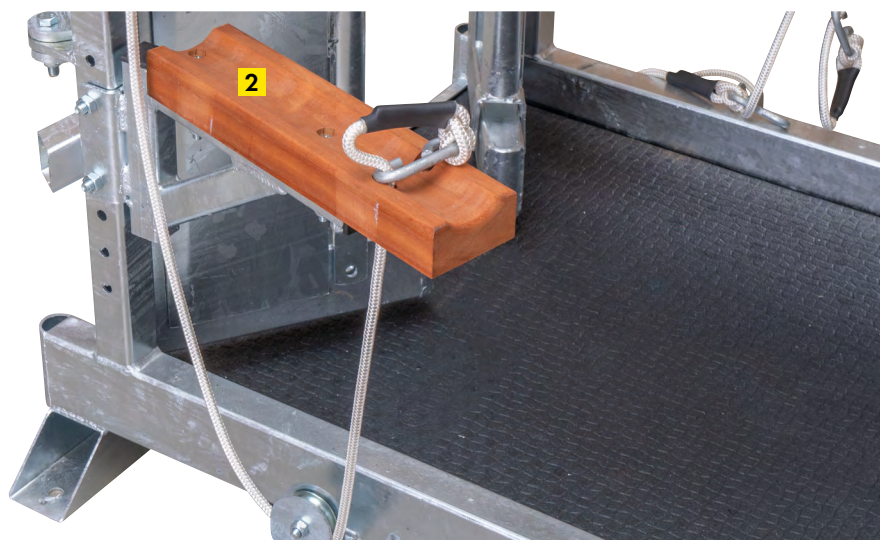
- 2 electric motors for front foot winches
- 2 electric motors for belly strap
- 1 electric motor for rear foot winch
- 1 electric motor for backstop/push bar
- 230 V power supply
- 1 600 W per electric motor
- 1500 kg lifting force each motor
- 2 power outlets, 1 LED lighting
- Completely hot-dipped galvanised
- Automatic headbail unit
- 20 mm thick rubber mat - less noise, non-slip
- Emergency stop switch for each winch

- The hoof care crush has a swivel arm. That arm can be rotated around the crush
- To open the backstop / push bar, the swivel arm has to be rotated out of the opening area of the backstop/push bar
- Equipment of the swivel arm:
  - 2 power outlets
  - 1 tool holder
  - 1 LED lighting
- For a safe standing position, the hoof care crush should be fixed on a concrete floor

## Before using:

1. • Front foot rests **2** are reversed for transport
  - Fit them on the opposite sides (use the 19 mm wrench)
  - They must show to the outside
2. Connect the hoof care crush **1** to 230 V power supply
3. To lower the front foot winches on both sides, press the button [↓]
4. To lower the belly strap **3**, press the button [↓]
5. Prepare the hook for rear foot treatment **4** (supplied loose)

## 1. Operating buttons

## 2. Capture of the animal

Headgate only for capturing hornless cattle!

- Wings of the headgate must be directed to the inside to enable the automatic function. The animal presses the doors with its shoulders and closes the gate in this way.
- If desired, adjust the neck width using bolts and levers.
  - » Neck width from 10 to 28 cm in 5 steps
  - » Operating of the headgate from left or right side
- After fixing the animal in the headgate, lower the backstop / push bar to additionally fix the animal.
- Press the button [↑] to move the belly strap **3** upwards, until the belt will touch the belly loosely.

## 3. Hoof care treatment

**A**

Front feet:

- Place the strap around the foot, press button [↑] to move the foot upwards and place it on the foot rest **2**.
- Place the swivel arm in the correct position, turn the light on, connect your tools.
- After finishing: press the button [↓] and release the strap.

## Mounting Instructions

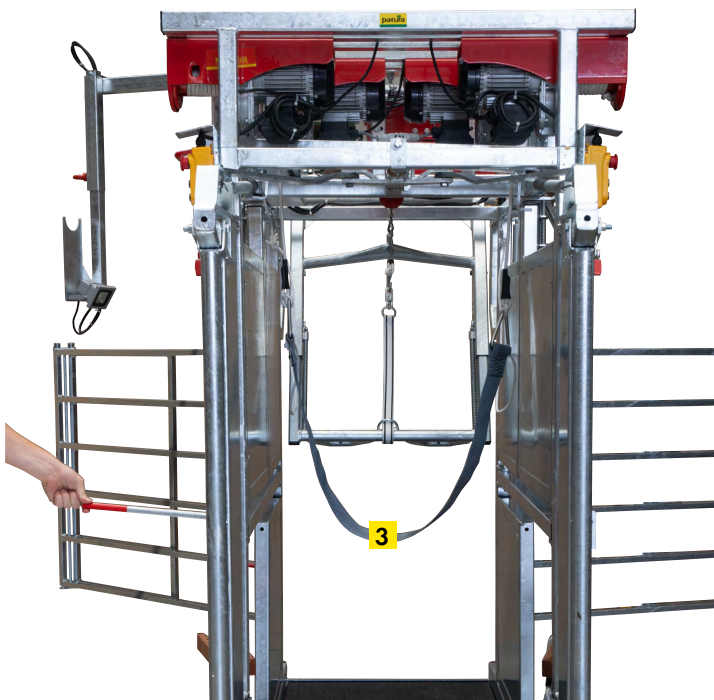
### B Rear feet:

- Insert the rope between the straight and bent bar.
- Place the hook **4** above the ankle joint and attach it to the hook of the rope.
- Press the button [↑] and lift the hind leg - the leg slides automatically into the bend of the curved bar and is fixed there.
- After finishing, press button [↓] and release the hook **4** from the foot.

### 4. Release of the animal

- Lower the belly strap **3**.
- Push the lever of the headgate upwards - Now the animal can open the doors on its own.
- If there are some problems for the animals to open the wings, use the lever on the side.
- After that, the headgate automatically goes back in the catch position.

## 5. Other applications

- All side doors can also be opened for vaccinations or other treatments.
- The central crossbar can be removed for better access to the animal.

## Advantages:

Backstop and rear foot winch

Automatic headbail unit

Power outlet and LED lighting

20 mm thick rubber mat

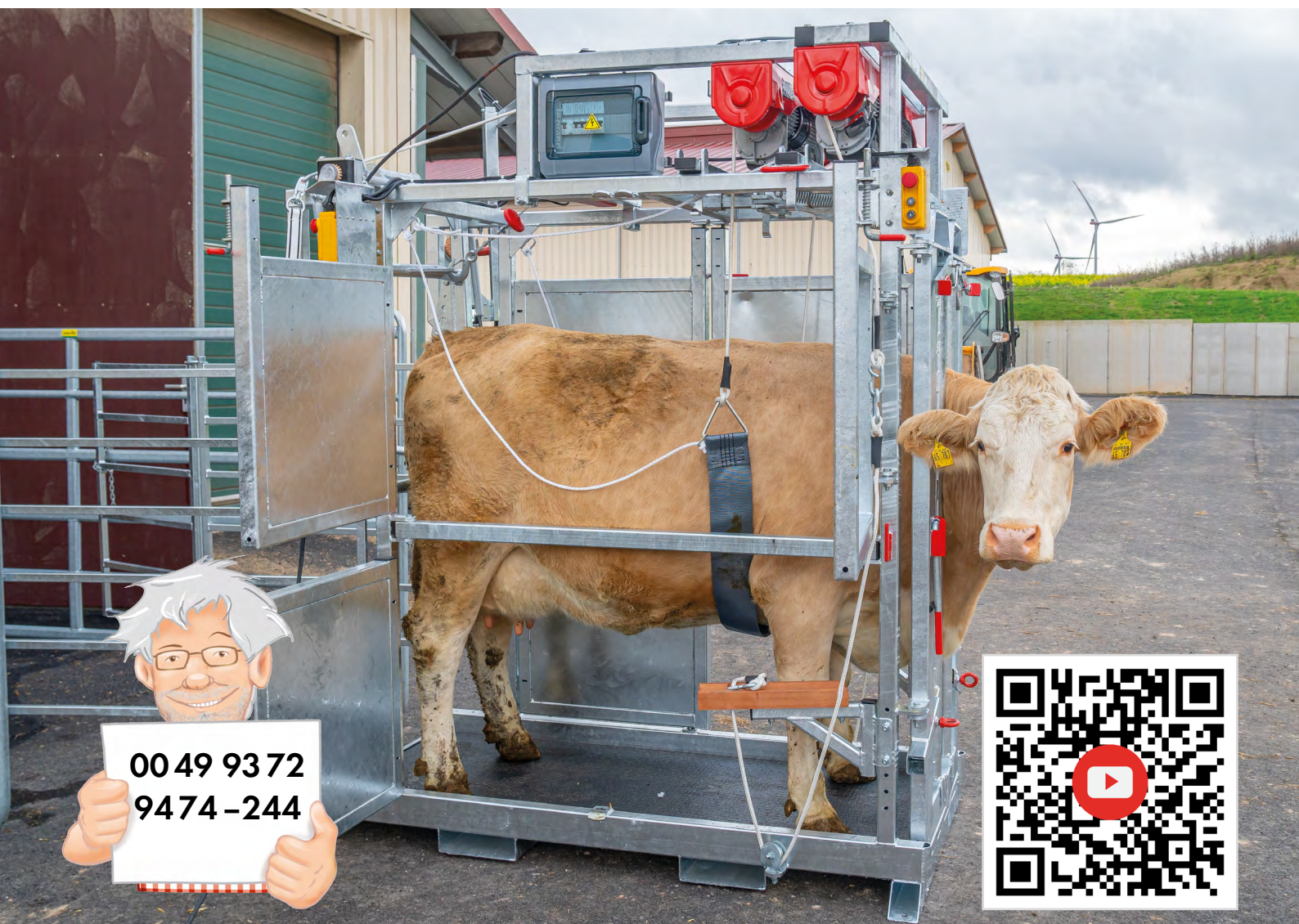
Comfortable hoof care thanks to electric winches

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